


Infrared RF Vacuum Cavitation Roller Slimming Machine



Cellulite Killer

- Non-Invasive • Easy to Use
- No Downtime • Comfortable

Treatment Principle

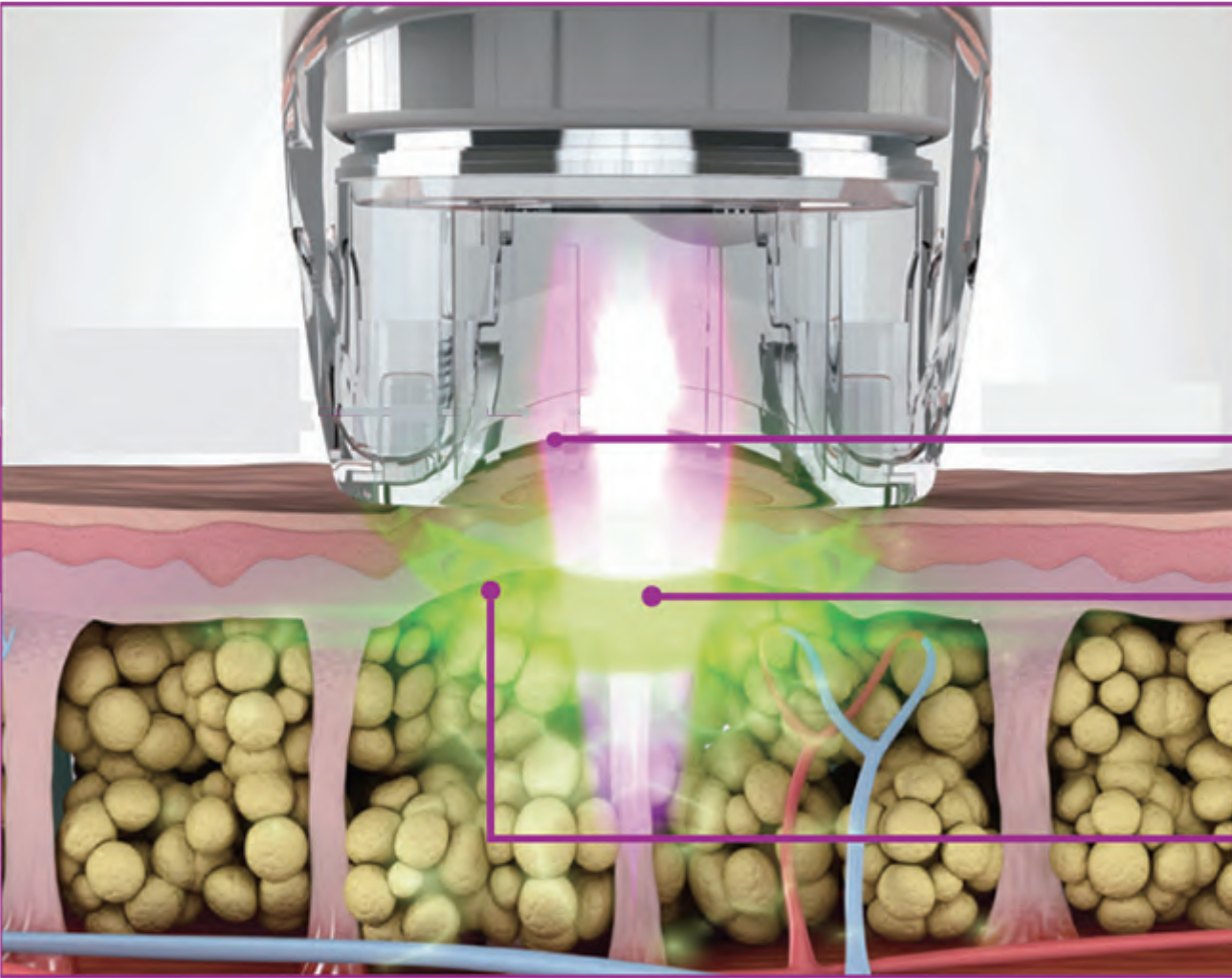

A vacuum elevates the target tissue, bringing it closer to the energy source.


Infrared light (IR) directly heats the targeted area.

Bi-polar radio frequency (RF) energy safely guides heat evenly throughout the targeted area, so that it heats faster without damaging the skin.

Scope of Treatment

- Body Contouring • Cellulite Removal • Body Slimming • Skin Texture & Tone
- Wrinkle Removal • Face Lifting • Skin Tightening • Circumference Reduction

Before & After

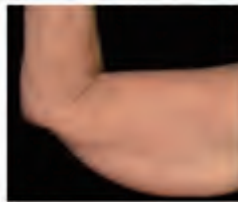


What is cellulite?

Cellulite is a structural skin change that occurs in most post-pubertal females whether they are thin or overweight. It presents itself as a modification of skin morphology evident by skin dimpling and nodularity that occurs mainly in women on the pelvic region, lower limbs, and abdomen, and is caused by the herniation of subcutaneous fat within fibrous connective tissue, leading to a padded or orange peel-like appearance. Cellulite can occur in women of all ages and tends to become more visible with age. Over time, the skin loses its thickness and thereby enhances the visibility of the unsightly cellulite bumps.



How does Infrared RF Vacuum Roller work to help get rid of cellulite?

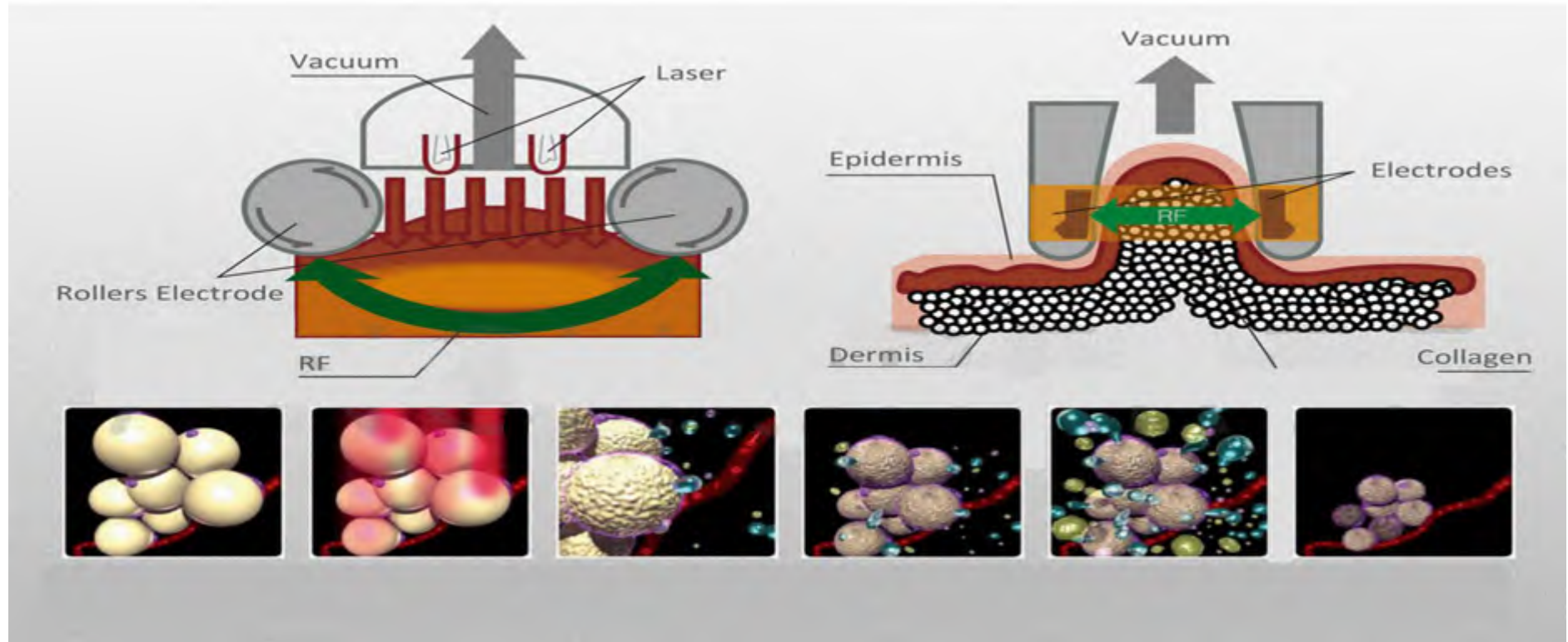


The Infrared RF Vacuum Roller combines four different technologies including IR (infrared), Bi-Polar RF (radio-frequency) and mechanical tissue manipulation using pulsed vacuum and massage rollers. The combination of the IR and vacuum coupled RF technologies causes deep heating of the connective tissue including the fibrous septae which in turn promotes an increase in collagen depositing and local cellular metabolism resulting in a localized reduction in skin laxity and volume. The additional mechanical tissue manipulation of the Vela causes an immediate increase in circulation and lymphatic drainage, both essential components for healthy skin structure.

Advantages

- 5 Technologies in one machine - Vacuum + 40khz cavitation+ 940nm Near-Infrared Laser + RollersBipolar RF
- Profitable business driver - high patient demand for cellulite & body contouring treatments
- Add more patients to your practice with short treatment sessions
- Excellent results in just 1-3 sessions
- Enhanced user experience with new Guided Mode software
- Short treatment sessions and outstanding results, from as little as a single session
- Zero downtime and little or no discomfort
- High patient satisfaction




Four Different Treatment Handpieces for Whole Body Treatment.

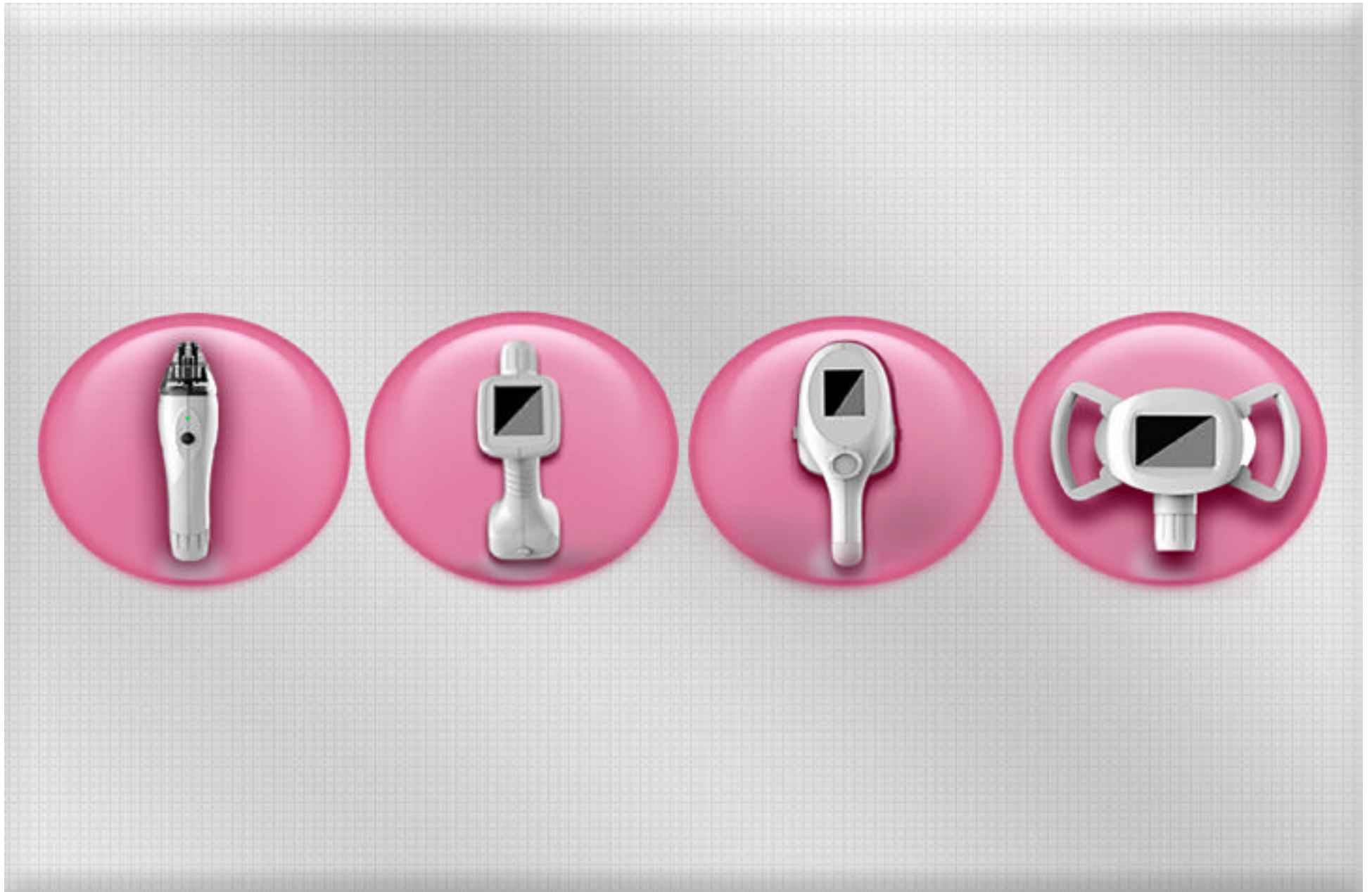


Roller Big




Roller Middle

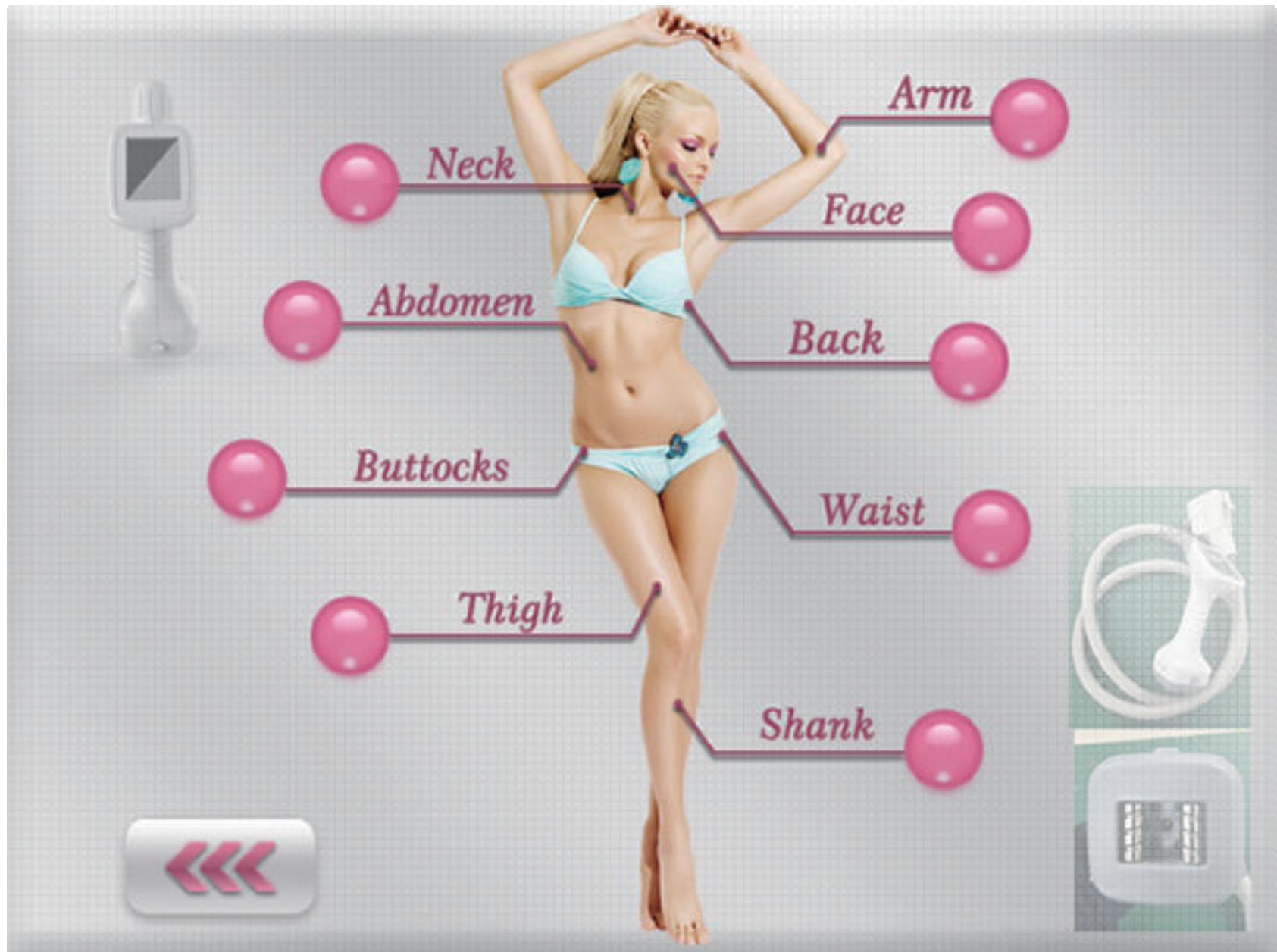

Vacuum cavitation

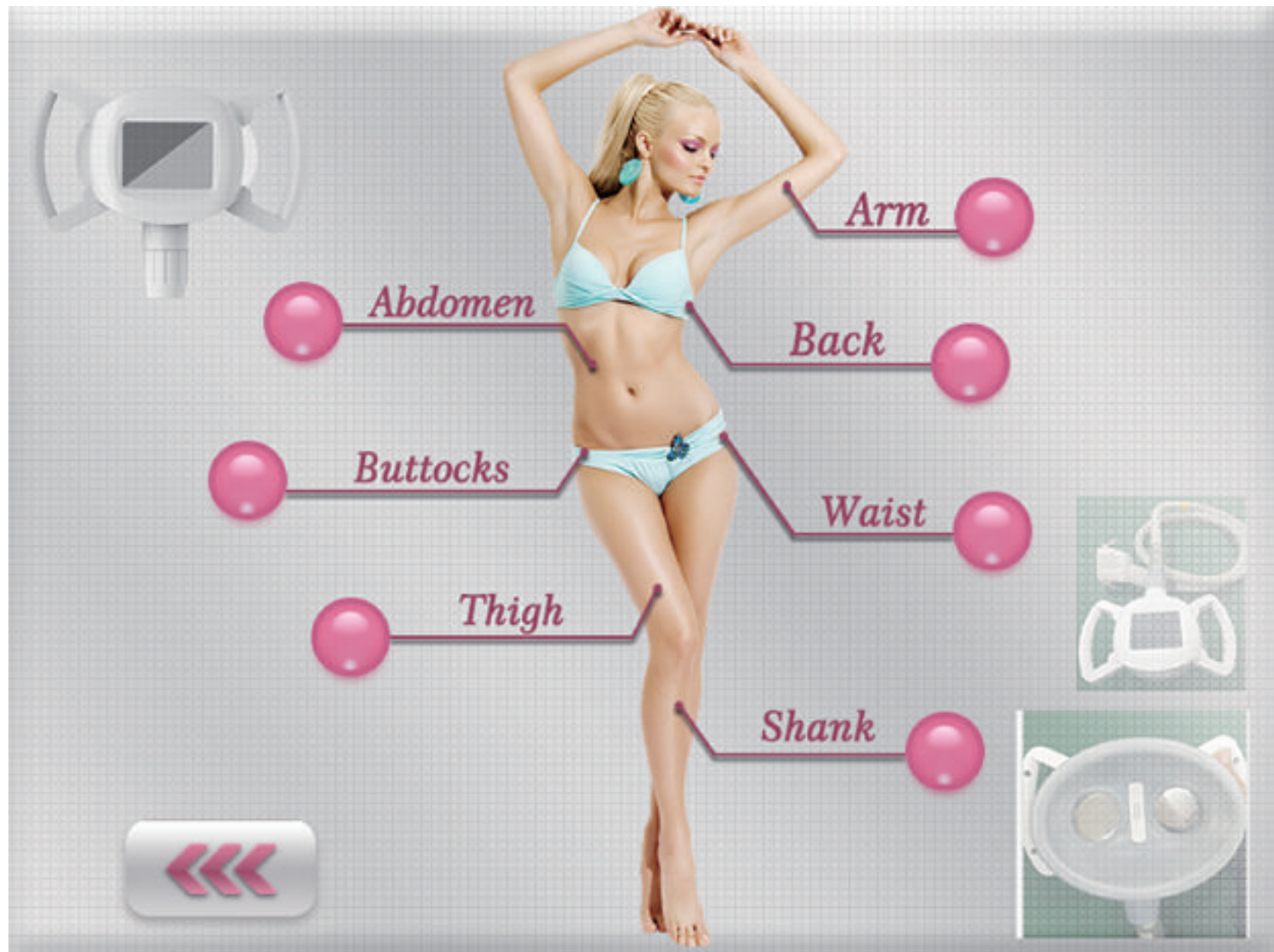

RF small

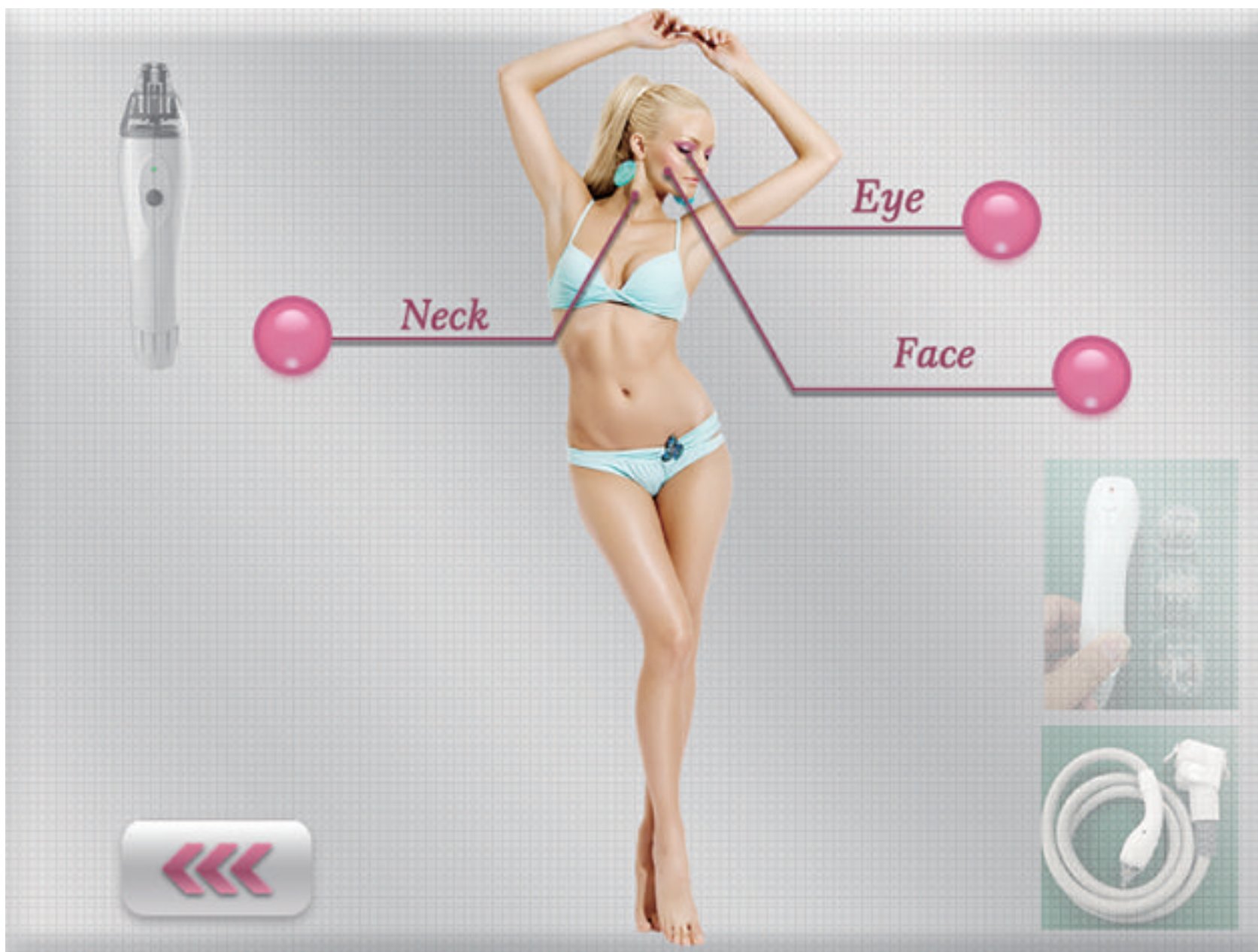

Specification

Power	1200w
Radio Frequency power	400w
Infrared power	5-20w
Infrared wavelength	940NM
Size of treatment handle	(Roller Big)40mmx66mm (Roller Middle)30mmx44mm (Cavitation)90mmx120mm (RF small)4.5mmx9mm 9mmx13mm;16mmx25mm
Vacuum mode	Continuous
RF frequency	2Mhz
Life of handle	500 hrs
Vacuum level	1-8 adjustable
Cavitation frequency	40khz
Cavitation power	600w
Roller Speed	0-36r /m
Voltage	AC220V±10%,10A;50HZ/ AC110V±10%10A,60HZ

WHY CHOOSE US

INCREASE DEMAND



Increased demand for non-invasive treatments

TRAIN CERTIFICATION



Full Training and Certification for 4 people valued at \$1500

STAY AHEAD OF THE MARKET



Keeping ahead of the market with latest Body Shaping Treatments for Men and Women

SUPPORT



Professional service team with 24hours online service (Phone support)

WARRANTY



3 years warranty (except consumables), with whole life free service.

PROFESSIONAL MANUFACTURER



More than 10 years manufacturer experience

TRAINING



User manual, operation video and online training through webcam for free

YOUR OWN BRANDING



Put your logo on the machine and screen if needed.

Who would be an ideal candidate for this treatment?

The treatment is recommended for normal to overweight patients who suffer from unsightly cellulite in the pelvic region, around the hips, abdomen, or lower limbs. For optimized results, these patients should also be committed to maintaining a healthy lifestyle. The treating clinician has the final determination for choosing the best candidates for the treatment.

How many treatments needed?

For body and limbs, 8-10 treatments each session, each 4-5 days for one treatment, 30 minutes each treatment.

For face, 10 treatments one session, once a week, 15-20 minutes each treatment.

For eye wrinkle, 10 treatments one session, once a week, 15 minutes each treatment.

How long the results will last?

We recommend 10 treatments as a session, there will be improvements differently after each treatment. Results can last up to a few years depending on age, lifestyle and hormonal changes, maintenance sessions are recommended to extend the effect of the results, we recommend once a month after desired results have been obtained. 1-5cm will be reduced on thighs, 2-6cm on abdomen and waists.

What is the treatment procedure like?

Most patients find the treatment comfortable and describe the treatment as feeling like a warm deep-tissue massage. The treatment parameters are easily adjusted to ensure a comfortable treatment experience. It is normal to experience a warm sensation for a few hours after your treatment. Some patients report a pinkish appearance at the treatment area that may last a few hours post treatment, and skin may appear pink for several hours.

What will happen after treatment?

- 1) Urinary output will be increased.
- 2) feel thermal for 30minutes to an hour.

What should be noticed after treatment?

Please avoid sunbath and tanning treatment.

Who can operate this machine?

Any trained operator can use it.

Applications

- 1) Neck lift, face lifting, improve crow's feet, dark circles, eye bags, etc.
- 2) Cellulite reduction for arms, legs, back, waists, abdomen.
- 3) Body shaping after childbirth.
- 4) Striae gravidarum reduction.

What are the contraindications ?

There are no major contraindications for the treatment. For patients with pacemakers/de-fibrillator, pregnant or nursing or severe health problems, it is recommended to defer to their physician prior to commencing treatment.

About us

Beijing Leadbeauty International S & T Co., Ltd is the professional manufacturer of beauty equipment medical equipment, Beauty Salon products and so on.

Our company was built in 2003, we have our own research and development department and our own factory so we can provide OEM and ODM service for the distributors all over the world. In beauty equipment industry, our factory is one of the biggest one in China, it includes several production lines, Material Library, shipping department, Inspection Area. We prove that no failure products sent to our customers.

Contact us

Beijing Leadbeauty International S & T Co., Ltd.

E-mail : Leadbeauty@leadbeauty.net

Whats app: +86 13269056123

Website: www.leadbeauty.net

ADD: West zone building No.5 Mauhwa Industry Park, No.1 Caida 3rd Street Shunyi District Beijing 101300 China